



BioColors Fitting Set and BioColors/BioSport/BioMed Lens Cleaning Recommendations

BIOCOLORS OR BIOMED/BIOSPORT FITTING SETS:

It is recommended that the lenses be cleaned using Lens Fresh Extra Strength cleaner OR Clear Care between trialing with patients. Keep in mind Clear Care solutions are hydrogen peroxide based and are not recommended for patient use once they receive their final contact lenses as routine daily use will eventually fade the lens color. Once the lens(es) have been cleaned with either Lens Fresh or Clear Care, store the lenses back in the vial in fresh BioTrue multipurpose solution. If the diagnostic tool or tweezers are used, clean with an alcohol solution or swab and let air dry before storage.

A pair of white, rubber tipped tweezers are sent with each fitting set. Please ensure prior to using tweezers both rubber tips are on the end of the tweezers to prevent tearing the lenses with use. Only insert one side of tweezers into vial to slide out the contact lens. Tweezers may be using with the BioColors Fitting set to aid with stacking lenses on the Diagnostic Tool when color matching a patient's eye for a prosthetic lens design.

Lens Fresh Use Instructions:

Prior to Insertion:

- 1) Thoroughly wash hands with soap and warm water.
- 2) Remove lens from vial and place in palm of hand.
- 3) Apply 1-2 drops of Lens Fresh to both surfaces of lens.
- 4) Gently rub lens with forefinger of opposite hand for 15-20 seconds.
- 5) Rinse thoroughly with sterile saline or multipurpose solution
- 6) Lens is ready for insertion.

After Removal and Storage:

- 1) Thoroughly wash hands with soap and warm water.
- 2) Remove lens and place in palm of hand.
- 3) Apply 1-2 drops of Lens Fresh to both surfaces of lens.
- 4) Gently rub lens with forefinger of opposite hand for 15-20 seconds.
- 5) Rinse thoroughly with sterile saline.
- 6) Place lens in correct vial and fill with disinfecting storage solution.
- 7) Lens can be stored until needed for next patient. Repeat steps for insertion for each patient when using BioColors Fitting Set.

****NOT FOR USE IN THE EYE. RINSE LENSES THOROUGHLY. IF BURNING OCCURS, PLEASE REMOVE LENSES AND RINSE AGAIN. DO NOT SOAK LENSES IN LENSFRESH****

Please contact Consultation at 1-866-300-6257 with questions or for assistance.

A complimentary bottle of Lens Fresh and a pair of tweezers are provided with your fitting set and should be retained for future use. Additional bottles of Lensfresh may be purchased from Orion Vision Group.

Patients may purchase Lensfresh directly through www.lensfresh.com

Please see attached MSDS for further information on Lens Fresh.